

# Detartrine

## Dental scaling / polishing

### PROPERTIES

DETARTRINE is available as ready-to use paste designed for dental scaling and polishing.

**DETARTRINE** is pink paste, with a pleasant taste. It has mechanical abrasive action due to the presence of silica, which makes it possible to complete the removal of dental tartar, after using an ultrasonic or hand scaler, without causing erosion of dental enamel.

DETARTRINE contains zirconium particles. These particles look like flat scrapers, the sides of which are separated by a very hard edge. When this paste is rubbed against a rough surface, the particle edges shave the bumps off.

The essential oils incorporated into DETARTRINE give a feeling of freshness, which is very much appreciated by the patients. In addition, they have a slight balsamic effect: they relieve the irritation that mechanical scaling might cause on sensitive gums.

### INDICATIONS

- Finishing of scaled surfaces,
- Removal of food and tobacco stains,
- Fine polishing of fillings.

### INSTRUCTIONS FOR USE

Take the required amount of paste with a spatula and place it onto a polishing wheel.

**Note:** Mouth rinsing can be performed during or at the end of scaling.

### PRESENTATIONS

45 g jar

**For professional dental use only**

Manufactured and Marketed by:

**SEPTODONT HEALTHCARE INDIA PVT.LTD**

C- 1/2, MIDC. Taloja, Tal: Panvel,  
Dist: Raigad - 410 208, Maharashtra, INDIA.

Customer Care No.: 022 27401970/71

email: info@septodontindia.com



DO26108150

### PLEASE CHECK THESE SPECIFICATIONS CAREFULLY

DT-LF/10/2023/37



A 705, TTC Industrial Area,  
MIDC Mahape  
Navi Mumbai 400 710 India  
+91 9867134433  
hitechgraphics@printingindia.com

Job : Detartrine Leaflet  
Date : 11 October 2023  
L x W x H : 80 x 215 mm  
Printing : Black  
Leaflet Type : Insert  
 Die Cutting  Foiling

Paper : 60 gsm Maplitho  
Folding Size : 40 x 48 ± 1 mm

#### Colours

Black

Keyline

#### Finishing :

Varnish  Full  Gloss

Aqueous  OPZ  Matt

Lamination

Foil

**Size as per your submitted Dummy**

Approved by \_\_\_\_\_ Date : \_\_\_\_\_

Signature for approval and agreement to statements 1- 5  
above : \_\_\_\_\_

1. Hitech Graphics requires a signed proof approval prior to printing. Please email your approval or changes to hitechgraphics@printingindia.com
2. The colours you see on your screen or on a colour proof will be different than the actual colours printed. PMS colours will be matched to the actual Pantone book and may vary slightly depending on the material used. If you have exact colours or finishing effects in mind, we recommend coming in while the cartons are on press to approve. Please tick wherever applicable.
3. It is your responsibility to examine this proof carefully for layout, size, text, graphics, etc. Please reconfirm that the finished label configuration will work with your applicator and will fit as expected on your product.
4. Our terms require payment prior to shipping unless prior credit terms have been established. Cancellation of your order after proof approval will incur charges.