# Canal +

## COMPOSITION

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# Canal +

## COMPOSITION

Water-soluble lubricant, edetic acid, carbamide peroxide.

## PROPERTIES

- lubricates the root-canal and facilitates endodontic instrument penetration,
- makes shaping more efficient,
- due to its effervescent properties makes cleaning of the root-canal easier.

## INDICATION

Facilitates root-canal preparation.

## CONTRAINDICTION

Hypersensitivity to any component.

## INSTRUCTIONS FOR USE

- 1) Coat the instrument to be used with the gel for lubrication;
- 2) Shape to the desired number;
- 3) Irrigate with a sodium hypochlorite solution.

## OPERATIVE TECHNIQUE

### 1. Pulp chamber

- Isolate the tooth to be treated, using a rubber dam;
- Open up the pulp chamber in the usual manner;
- Fit the applicator tip to the CANAL PLUS syringe;
- Place CANAL PLUS;
- Irrigate with a sodium hypochlorite solution to remove pulp debris and clear the entrance to the root-canal(s);
- Remove the liquid from the chamber by aspiration.

### 2. Root-canal

The way CANAL PLUS is used will vary, depending on the anatomical features of the root-canal(s) to be prepared. Incorrect direction of reaming should be avoided in any instance. For preparation of wide and relatively straight root-canals, CANAL PLUS can be used starting with the first instrument number. For narrow, calcified and/or irregular shaped canals it may be necessary to assess the shape and working length and to start reaming up to a rather high number before applying CANAL PLUS. The instruments should be pre-curved and adjusted (stopped) to facilitate the work.

Then proceed by coating the instruments with CANAL PLUS, working in the usual way; then rinse with hypochlorite, dry and fill the canal as normal.

## Notice

Avoid using CANAL PLUS right up to the last numbered reamer. Although it is highly water soluble, always make sure no trace of CANAL PLUS remains in the canal. CANAL PLUS should not be left in the tooth between visits.

## SPECIAL WARNINGS AND PRECAUTIONS FOR USE

Causes serious eye damage.  
Refer to safety data sheet.

## STORAGE

Stored in a cold place.

## PRESENTATIONS

5 g syringe + 5 applicator tips.

*For professional dental use only*

Manufactured and Marketed in India by  
Septodont Healthcare India Pvt. Ltd  
C-1/2, MIDC, Talaja, Tal. Panvel  
Dist. Raigad - 410 208, Maharashtra (India)  
Customer Care No. : 022-2740 1970 / 71  
email : info@septodontindia.com

C130108230

PLEASE CHECK THESE SPECIFICATIONS CAREFULLY		CP Lf/08/2023/24
 A 705, TTC Industrial Area, MIDC Mahape Navi Mumbai 400 710 India +91 9867134433 hitechgraphics@printingindia.com	Job : <u>Canal+ Leaflet</u>	Paper : <u>NLT 50 gsm</u>
	Date : <u>24 Aug 2023</u>	Folding : <u>52 x 80 ± 1mm</u>
	L x W : <u>210 x 80 mm</u>	
	Printing : <u>Black</u>	
	Leaflet Type : <u>Insert</u>	
	<input type="checkbox"/> Die Cutting <input type="checkbox"/> Foiling	
		Colours <input checked="" type="checkbox"/> Black
		Finishing : <input type="checkbox"/> Varnish <input type="checkbox"/> Full <input type="checkbox"/> Gloss <input type="checkbox"/> Aqueous <input type="checkbox"/> OPZ <input type="checkbox"/> Matt <input type="checkbox"/> Lamination <input type="checkbox"/> Foil Size as per your submitted Dummy
		<input type="checkbox"/> Approved by _____ Date : _____ Signature for approval and agreement to statements 1- 5 above : _____
<p>1. Hitech Graphics requires a signed proof approval prior to printing. Please email your approval or changes to hitechgraphics@printingindia.com</p> <p>2. The colours you see on your screen or on a colour proof will be different than the actual colours printed. PMS colours will be matched to the actual Pantone book and may vary slightly depending on the material used. If you have exact colours or finishing effects in mind, we recommend coming in while the cartons are on press to approve. Please tick wherever applicable.</p> <p>3. It is your responsibility to examine this proof carefully for layout, size, text, graphics, etc. Please reconfirm that the finished label configuration will work with your applicator and will fit as expected on your product.</p> <p>4. Our terms require payment prior to shipping unless prior credit terms have been established. Cancellation of your order after proof approval will incur charges.</p>		