

One Side Printing

Cimpat Rose

Temporary Dental Cement

PROPERTIES

**Paste for temporary filling. - adheres to dentin,
- hardens rapidly in the cavity.
- expands slightly while setting.
- ensures tight sealing.**

WARNINGS AND PRECAUTIONS FOR USE

Like any filling material, Cimpat Rose remains in place only when the cavity is properly shaped. Cimpat Rose may be softened during setting by certain solvents contained in liquids or pastes used for root canal treatment. Therefore after any root canal treatment and before filling with Cimpat Rose, it is required to thoroughly clean the cavity walls and allow them to dry.

For easy handling, use a warm spatula.

T-Toxic

N - Dangerous for the environment Contains dibutyl phthalate (DBP)

Very toxic to aquatic organisms, may cause long term adverse effects in the aquatic environment, Irritating to eyes. May cause harm to the unborn child. Possible risk of impaired fertility. Wear suitable protective clothing and gloves. In case of insufficient ventilation, wear suitable respiratory equipment. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

PRESENTATION

CIMPAT ROSE
36 g jar

Manufactured and Marketed by
SEPTODONT HEALTHCARE INDIA PVT LTD
C-1/2, MIDC, Taloja, Tal. Panvel,
Dist.: Raigad-410208, Maharashtra (India)
Customer Care No.: 022-27401970/71
info@septodontindia.com

M.L. MFG/MD/2022/000720



C114I01230



Customer : SEPTODONT HEALTHCARE	Location : NA	Date : 03-Mar-23	
Product Code : C114I01230	Market : NA	Pharma Code :	
Product Name : Septodont_Cimpat_Rose_Insert_LF			
Dimension : 80 x 215mm	Carton Style : Insert		
Substrate : NA			
Surface Finishing : NA			
Colours Front : K			
Non Printing Colours :			
Note- This approval will be considered for the final printing. Please recheck for corrections indicated earlier, in this proof also.			
Prepared By	Checked By	Profile Checked	Approved By
Umesh			