

R 4

Chlorhexidine digluconate based solution
Root-canal solution

COMPOSITION

Chlorhexidine digluconate (20 % solution)0.2 gm
Denatured ethyl alcohol40.0 gm
Excipients q.s.100.0 gm
(acetic acid, purified water)

PROPERTIES

R4 is a solution for the cleaning of root canals by a final irrigation. The presence of chlorhexidine digluconate permits the bacterial disinfection of canals, because of its specific action on anaerobic strains in the root-canal flora and endo-periodontal lesions.

INDICATIONS

Root-canal irrigation when endodontic re-treatment is required and/or when root-canal treatment requires several sessions due to infected canal.

CONTRA INDICATIONS

Hypersensitivity to any component.

INSTRUCTIONS FOR USE

After chemical and mechanical root-canal preparation, flush the canal with sterile water or an EDTA solution. Dry the canal with paper points. For the final irrigation, use R4 solution in a disposable pipette, preventing extrusion of the solution beyond the apical foramen. Do not flush. Remove the excess solution which could adversely affect the sealing process of the temporary crown restoration. If necessary, place a cotton pellet in the access cavity and seal it hermetically with a temporary filling, ensuring that the cotton pellet is completely covered. Use a new pipette each time you take some solution.

At the next session (after 24 hours to 15 days), flush the canal with sterile water or an EDTA solution. Proceed with the chemical and mechanical preparation of the root-canal and/or fill the cavity with a permanent filling, depending on the clinical situation and according to routine procedures.

WARNINGS AND PRECAUTIONS FOR USE

When irrigation associates sodium hypochlorite and chlorhexidine, rinse with water between the two irrigants to avoid any interaction and precipitate formation.

Danger.

Highly flammable liquid and vapour.

Causes serious eye irritation.

Refer to safety data sheet (available upon request).

STORAGE

Close container tightly after each use.

Use within 6 months after opening.

Keep in a cool place (10 to 25°C).

PRESENTATION

13-ml bottle

For professional dental use only

Manufactured and Marketed in India by :

SEPTODONT HEALTHCARE INDIA PVT. LTD.

C - 1 / 2, MIDC, Taloja, Taluka - Panvel,

Dist. Raigad - 410 208, Maharashtra, INDIA

Customer Care No. : 022-2740 1970 / 71

E-mail : info@septodontindia.com



R102110200

PLEASE CHECK THESE SPECIFICATIONS CAREFULLY

R4-LF/03/2024/47



A 705, TTC Industrial Area,
MIDC Mahape
Navi Mumbai 400 710 India
+91 9867134433

hitechgraphics@printingindia.com

Job : R4 - Leaflet Front

Date : 12 March 24

L x W : 80 x 215

Printing : Black

Leaflet Type : Insert

Die Cutting Foiling

Paper : 60 gsm Maplitho

Folding : 40 x 48 ± 1mm

Colours

Black

Finishing :

Varnish Full Gloss

Aqueous OPZ Matt

Lamination Foil

Size as per your submitted Dummy

Approved by _____ Date : _____

Signature for approval and agreement to statements 1- 5
above : _____

- Hitech Graphics requires a signed proof approval prior to printing. Please email your approval or changes to hitechgraphics@printingindia.com
- The colours you see on your screen or on a colour proof will be different than the actual colours printed. PMS colours will be matched to the actual Pantone book and may vary slightly depending on the material used. If you have exact colours or finishing effects in mind, we recommend coming in while the cartons are on press to approve. Please tick wherever applicable.
- It is your responsibility to examine this proof carefully for layout, size, text, graphics, etc. Please reconfirm that the finished label configuration will work with your applicator and will fit as expected on your product.
- Our terms require payment prior to shipping unless prior credit terms have been established. Cancellation of your order after proof approval will incur charges.