(FR)**COMPOSITION**

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(EN)**COMPOSITION**

Ethyl acetate, Amyl acetate and thymol.

PROPERTIES

Due to its composition, this solvent can soften both traditional zinc oxide eugenol cements and phenolic resin type sealers

INDICATIONS

Root canal sealers solvent when canal retreatment is required

CONTRA INDICATION

Hypersensitivity to any component of the formulation.

ADVERSE EFFECTS

Allergic reaction

INSTRUCTIONS FOR USE

The operative field should be completely isolated: the use of a rubber dam around the tooth is recommended, as in all root-canal therapy. Mechanically clean out the pulp chamber and the entrance to the canals. Place several drops of ENDOSOLV in an adapted content before placing a drop of ENDOSOLV into the chamber. Clear the canal, using first a probe, then a broach, always dipping the point of the instrument in ENDOSOLV before inserting it in the canal. The instrument must be removed from the canal and wiped as soon as resistance is felt. Repeat the operation several times, until reaching the apex. Intermediate irrigation with sodium hypochlorite is effective in neutralizing the medium.

WARNINGS AND PRECAUTIONS FOR USE

After using ENDOSOLV to remove the old filling material, it is essential to prepare the tooth again and to carefully dry it, particularly when using a zinc-oxide eugenol cement as the new filling material.

Contains ethyl acetate
Highly flammable liquid and vapour
Causes serious eye irritation
May cause drowsiness or dizziness
Refer to safety data sheet (available upon request)

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